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Blood Glucose Monitoring System

Blood Glucose Meter **User Manual**

Safe AQ Smart, Safe AQ Voice, Safe AQ Air

Changsha Sinocare Inc.
No. 265, Guyuan Road, Hi-Tech Zone, Changsha,
Hunan Province, 410205, People's Republic of China
E-mail: intl_mkt@sinocare.com Website: www.sinocareintl.com
Customer service: +86-731-89935929

Shanghai International Holding Corp. GmbH (Europe)
Eiffelstraße 80, 20537 Hamburg, Germany.
Tel: +49-40-2513175 Fax: +49-40-255726
E-mail: shholding@hotmail.com
P/N: 36301102-B.5

CE 0123 For in vitro diagnostic use only

Sinocare®

4. Click "Link device"
5. Insert test strip, meter turn on.
6. If the bluetooth symbol displayed, it means connect successful. Perform test and your test result will be uploaded to your smartphone. If the bluetooth symbol is flash, please check if you have turned on bluetooth function on your smartphone and insert test strip again.

Note: Different App may be little different while pairing meter with smartphone.

IX. Control Solution Testing

Why Perform a Control Solution Test

- Ensures that your meter and test strips are working properly.
- Allows you to practice testing without using blood.

When Should You Perform a Control Solution Test

- When you begin using a new vial of Test Strips.
- When the vial cap is opened for a long time.
- When Test Strips have been exposed to extreme environmental conditions.
- When you want to check whether meter or test strips are work properly.
- When you drop or broken the meter.
- When you want to check whether the test steps are correct.

9

Important Information

- Use only Sinocare blood glucose control solution with your Meter.
- Check the expiration date on the control solution bottle. Do not use if expired.
- Use the test strips within a period of 6 months from the date that you first open the vial.
- Record the opened date on the control solution bottle, which is 3 months from when you first open it. Discard the bottle after 3 months.
- Discard any bottle that appears to be cracked or leaking.
- For in vitro diagnostic use only.

Performing a control solution test

Start with the blood glucose meter off. Prepare the following items before control solution test:

- a. blood glucose meter;
- b. blood glucose test strip;
- c. blood glucose control solution

STEP 1: Wash your Hands

Wash your hands with mild soap and water. Be sure to dry your hands before performing a test.

STEP 2: Insert a Safe AQ Test Strip

Insert a Safe AQ Test Strip with printing face up and the contact end into the test strip port until it cannot go any further. The blood glucose meter will turn on.

10